


An Introduction to Optune®

Here are some answers that may help you decide if Optune is right for you.

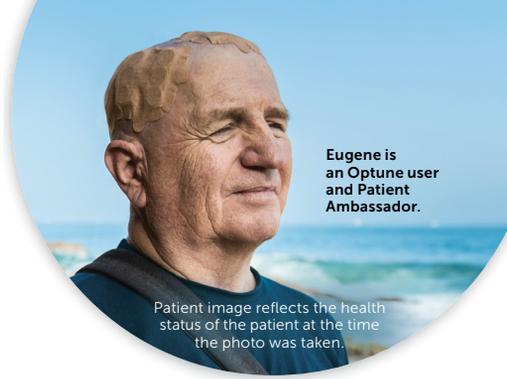

What is Optune?

Optune is a wearable, portable, **FDA-approved** treatment for adult patients with glioblastoma (GBM).

How would Optune fit into my treatment plan?

In newly diagnosed GBM, Optune is used together with chemotherapy* after radiation with chemotherapy.



In recurrent GBM, Optune is used alone after treatments like surgery and radiation.


What are the side effects?

Optune does not cause chemotherapy-related side effects.

- Mild-to-moderate skin irritation is the most common device-related side effect with Optune
- In a large clinical study, these side effects were typically manageable, reversible, and did not result in treatment discontinuation

In newly diagnosed GBM, Can Optune help me live longer?

In a large clinical study, people who added Optune to chemotherapy had a greater opportunity to live longer than those who had chemotherapy alone.

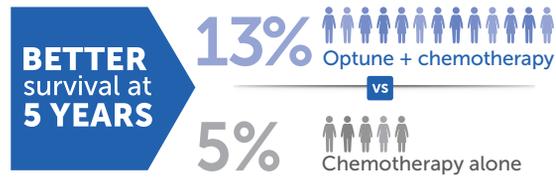

Nearly half of people (43%) on Optune plus chemotherapy were alive at 2 years, compared with 31% on chemotherapy alone.

How long should I use Optune each day?

You can get continuous treatment with Optune almost anywhere. If your doctor says it's okay, **you can use Optune while you go about your daily routine at work, while traveling, sleeping, and at social functions.**



For resources and additional information, visit Optune.com or call nCompass®:

1-855-281-9301 (toll free)

Connect with an Optune user: Optune Buddy Program:

1-844-247-1636 (toll free)

*Temozolomide, also known as Temodar®.

Please visit Optune.com/Safety for Optune Instructions for Use (IFU) for complete information regarding the device's indications, contraindications, warnings, and precautions.



Important Safety Information

What is Optune® approved to treat?

Optune is a wearable, portable, FDA-approved device indicated to treat a type of brain cancer called glioblastoma multiforme (GBM) in adult patients 22 years of age or older.

Newly diagnosed GBM

If you have newly diagnosed GBM, Optune is used together with a chemotherapy called temozolomide (TMZ) if:

- Your cancer is confirmed by your healthcare professional **AND**
- You have had surgery to remove as much of the tumor as possible

Recurrent GBM

If your tumor has come back, Optune can be used alone as an alternative to standard medical therapy if:

- You have tried surgery and radiation and they did not work or are no longer working **AND**
- You have tried chemotherapy and your GBM has been confirmed by your healthcare professional

Who should not use Optune?

Optune is not for everyone. Talk to your doctor if you have:

- **An implanted medical device (programmable shunt), skull defect (missing bone with no replacement), or bullet fragment.** Optune has not been tested in people with implanted electronic devices, which may cause the devices not to work properly, and Optune has not been tested in people with skull defects or bullet fragments, which may cause Optune not to work properly
- **A known sensitivity to conductive hydrogels** (the gel on the arrays placed on the scalp like the ones used on EKGs). When Optune comes into contact with the skin, it may cause more redness and itching or may rarely cause a life-threatening allergic reaction

Do not use Optune if you are pregnant or are planning to become pregnant. It is not known if Optune is safe or effective during pregnancy.

What should I know before using Optune?

Optune should only be used after receiving training from qualified personnel, such as your doctor, a nurse, or other medical staff who have completed a training course given by Novocure®, the maker of Optune.

- Do not use any parts that did not come with the Optune Treatment Kit sent to you by Novocure or given to you by your doctor
- Do not get the device or transducer arrays wet
- If you have an underlying serious skin condition on the scalp, discuss with your doctor whether this may prevent or temporarily interfere with Optune treatment

What are the possible side effects of Optune?

Most common side effects of Optune when used together with chemotherapy (temozolomide, or TMZ) were low blood platelet count, nausea, constipation, vomiting, tiredness, scalp irritation from the device, headache, seizure, and depression. The most common side effects when using Optune alone were scalp irritation (redness and itchiness) and headache. Other side effects were malaise, muscle twitching, fall and skin ulcers. Talk to your doctor if you have any of these side effects or questions.

Please visit [Optune.com/Safety](https://www.optune.com/safety) for the Optune Instructions for Use (IFU) for complete information regarding the device's indications, contraindications, warnings, and precautions.

For more information about Optune, visit [Optune.com](https://www.optune.com)

novocure®

©2023 Novocure GmbH. All Rights Reserved. Optune, nCompass, and Novocure are registered trademarks of Novocure GmbH. US-OPT-05523. January 2023.

 **OPTUNE®**
Elevate Expectations